


#122 PORTABLE HAND CONTROL

1-800-782-4335

www.HANDICAPSINC.COM

303-781-2062

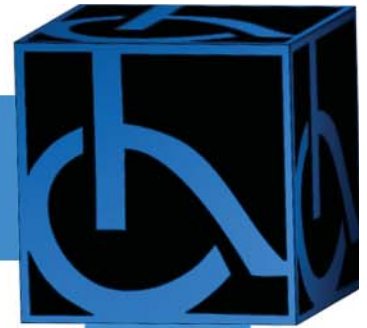

- ♿ **TO BRAKE:** USE LARGE KNOB AND PUSH STRAIGHT FORWARD TOWARD BRAKE PEDAL.
- ♿ **TO ACCELERATE:** HOLD GAS IN WITH THUMB, THEN SWING ENTIRE UNIT TO THE RIGHT, THIS WILL PUT LEVERAGE ON ACCELERATOR.
- ♿ **TO INSTAL:** USE WING NUTS TO TIGHTEN PEDAL BRACKETS. (2-15 minutes)
- ♿ **OPTIONAL VELCRO STRAP IS PROVIDED TO HOLD UNIT OUT OF THE WAY DURING TRANSFER.**

Warning: Brake rod and gas rod clamps must be tightened after adjustment with allen wrenches (supplied with control).

To be used on a **temporary** basis only. Must be used on vehicles with automatic transmission, power brakes & power steering. You must have unimpaired arm strength and dexterity to use them.

